

# THE WORRY-FREE CHOICE FOR EVERYONE, EVERYWHERE

As a healthcare professional, you have enough problems to worry about. Strikethrough, latex allergies and waste shouldn't be one of them.

Halyard Health created the STERLING\* Nitrile exam glove, a powder-free and natural rubber latex-free glove, to provide you with the protection, comfort, fit and sensitivity of latex, without the risk of exposure.

It's the one glove for all your non-surgical needs, facility-wide.<sup>1</sup>



**200 count box  
saves space  
and waste!**

## PATENTED NITRILE PROTECTION FROM STRIKETHROUGH

- Halyard's manufacturing process and patented formulation boosts the tensile strength of nitrile – the force required to break through the glove – to increase its already superior barrier protection qualities.
- With an Acceptable Quality Level (AQL) of 1.0, STERLING\* Nitrile Exam Gloves are 30% less likely to have critical defects than exam gloves that just meet Australian / New Zealand Standards for an AQL of 1.5.
- HALYARD\* STERLING\* Nitrile Exam Gloves have been cleared for use with chemotherapy, with no breakthrough up to 4 hours for 10 cytotoxic drugs.

## PERFORMANCE OF LATEX, IN A LATEX-FREE GLOVE


- STERLING\* Nitrile exam gloves are natural rubber latex-free, reducing the potential for Type 1 glove-associated allergic reactions to natural rubber latex proteins and a more comfortable feel.
- The process and formulation also allows for a thinner glove, providing the tactile sensitivity and dexterity of natural rubber latex, without the exposure.

## SAVE SPACE AND WASTE

- Standard length STERLING\* Nitrile Exam Gloves come in space saving 200 count dispenser boxes. That means up to a 50% saving in the storage space with fewer run-outs and fewer boxes to discard – and less waste for your facility.
- The SMARTPULL\* dispensing lessens your waste from gloves dropped or contaminated from fall-out.

### SMARTPULL\* DISPENSER BOX

SMARTPULL\* Dispenser incorporates two separate openings on the box. The first, smaller opening is used when the box is full and helps control dispensing to lessen waste from dropped gloves or multiple dispensing. When the box is half empty, the second, larger opening allows easier access to the gloves.

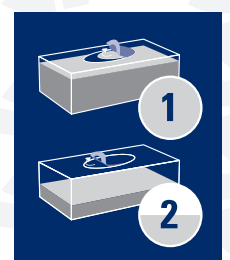

# MEET AND EXCEED THE STANDARDS

## KEY PRODUCT SPECIFICATIONS

- Exceeds AS/NZS 4011
- Powder-free
- Natural rubber latex-free
- Non-sterile
- Beaded cuff
- Enhanced wet and dry grip
- Ambidextrous
- Tested against 12 Chemotherapy agents

## PHYSICAL PROPERTIES

ELONGATION:	AS/NZS 4011:2014 <sup>1</sup>	STERLING* and STERLING* NITRILE-XTRA <sup>2</sup>
Before Ageing:	500%	650% ✓
After Ageing:	400%	500% ✓
TENSILE STRENGTH:		
Before Ageing:	7.0 N	7.0 N ✓
After Ageing:	6.0 N	6.0 N ✓

THICKNESS:	AS/NZS 4011:2014 <sup>1</sup>	STERLING* and STERLING* NITRILE-XTRA <sup>2</sup>
Middle Finger & Palm:	0.08	0.08 ✓
CUFF LENGTH:		
Minimum (mm):	240	242 ✓
Extended Length:	-	305 ✓

## CHEMOTHERAPY TEST RESULTS

### TESTED CHEMOTHERAPY DRUG AND CONCENTRATION

### AVERAGE BREAKTHROUGH DETECTION TIME

Cyclophosphamide (20.0 mg/ml)	No breakthrough up to 240 minutes
Doxorubicin HCl (2.00mg/ml)	No breakthrough up to 240 minutes
Etoposide (20.0 mg/ml)	No breakthrough up to 240 minutes
5-Fluorouracil (50.0 mg/ml)	No breakthrough up to 240 minutes
Paclitaxel (Taxol) (6.0 mg/ml)	No breakthrough up to 240 minutes
Cisplatin (1.0 mg/ml)	No breakthrough up to 240 minutes
Dacarbazine (10.0 mg/ml)	No breakthrough up to 240 minutes
Ifosfamide (50.0 mg/ml)	No breakthrough up to 240 minutes
Mitoxantrone (2.0 mg/ml)	No breakthrough up to 240 minutes
Vincristine sulfate (1.0 mg/ml)	No breakthrough up to 240 minutes
Carmustine (3.3 mg/ml)	Not for use with Carmustine
ThioTEPA (10.0 mg/ml)	Not for use with ThioTEPA



## HALYARD\* STERLING\* NITRILE EXAM GLOVES

Code	Description	Size	Packaging
13938	STERLING* Nitrile Powder-Free Exam Gloves	X-Small	200 eaches/box 10 boxes/case
13940	STERLING* Nitrile Powder-Free Exam Gloves	Small	200 eaches/box 10 boxes/case
13941	STERLING* Nitrile Powder-Free Exam Gloves	Medium	200 eaches/box 10 boxes/case
13942	STERLING* Nitrile Powder-Free Exam Gloves	Large	200 eaches/box 10 boxes/case
13943	STERLING* Nitrile Powder-Free Exam Gloves	X-Large	170 eaches/box 10 boxes/case

## HALYARD\* STERLING\* NITRILE-XTRA EXAM GLOVES

Code	Description	Size	Packaging
53137	STERLING* NITRILE-XTRA Exam Gloves	X-Small	100 eaches/box 10 boxes/case
53138	STERLING* NITRILE-XTRA Exam Gloves	Small	100 eaches/box 10 boxes/case
53139	STERLING* NITRILE-XTRA Exam Gloves	Medium	100 eaches/box 10 boxes/case
53140	STERLING* NITRILE-XTRA Exam Gloves	Large	100 eaches/box 10 boxes/case
53141	STERLING* NITRILE-XTRA Exam Gloves	X-Large	100 eaches/box 10 boxes/case



Infection Prevention Product Solutions  
 Halyard\* Education Foundation  
 Tools & Best Practice  
 Commitment to Excellence and Sustainability  
 Experienced Customer Support  
 Knowledgeable Sales Force  
 Innovation & Technology



O&M Halyard Australia Pty Ltd.

AUS: 1800 664 227

NZ: 0800 482 211

[www.halyardhealth.com.au](http://www.halyardhealth.com.au)

[www.halyardhealth.com.au](http://www.halyardhealth.com.au)



<sup>1</sup> AS/NZS 2014  
<sup>2</sup> Data on file.